vario **DIGITAL** 22 FHD Advanced

1659-22, 1659-221, 1659-222, 1659-223

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Operating instructions **EN**

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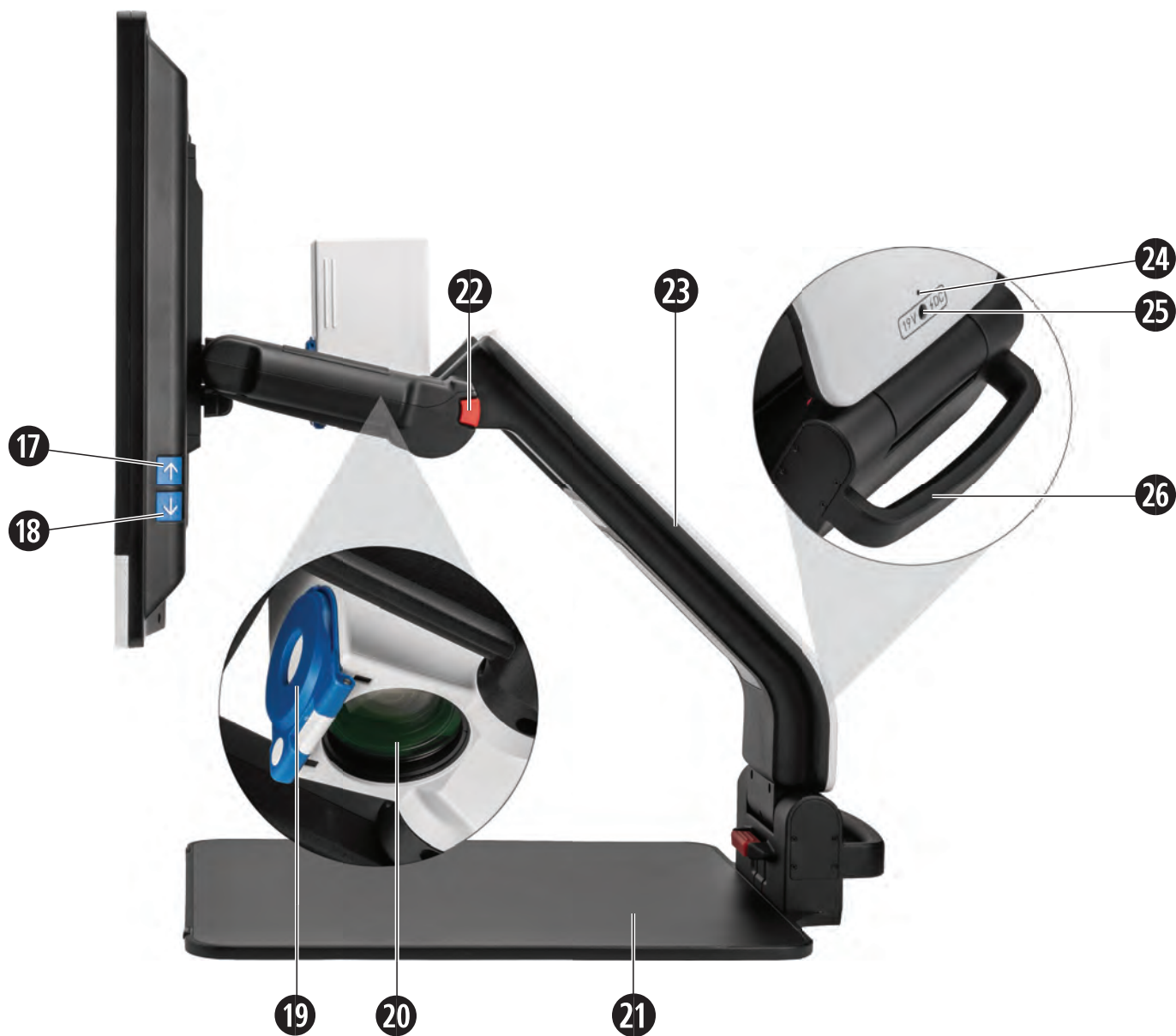
Manual de instrucciones **ES**

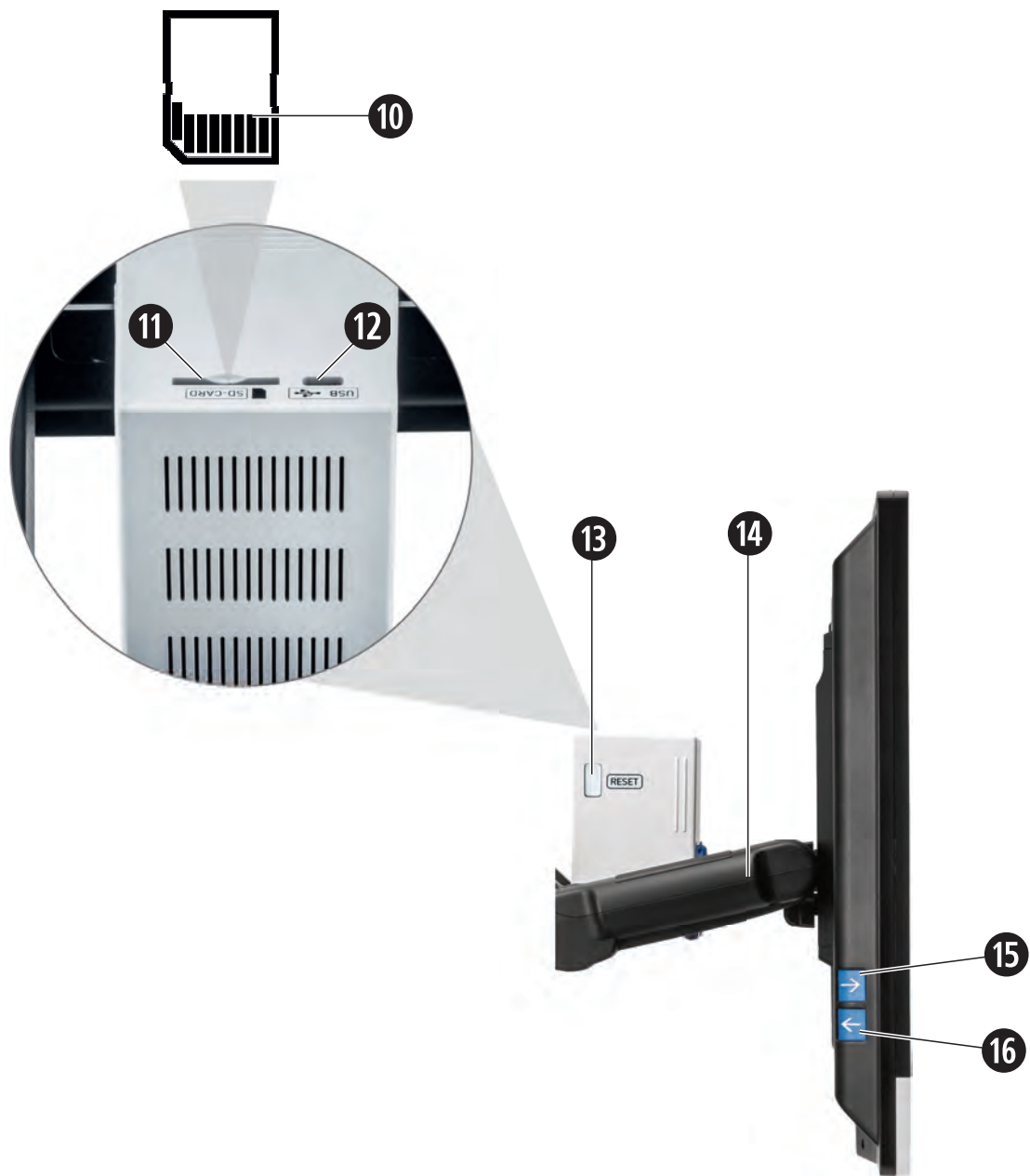
ESCHENBACH

Deutsch.....	1
English.....	15
Français	29
Italiano	43
Español.....	57

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Dear Customer,

Congratulations on the purchase of your Eschenbach vario DIGITAL 22 FHD Advanced.

As market leader in magnifying glasses and reading aids we at Eschenbach can look back on many years of experience, which is reflected in the quality, handling and design of our products.

Firmware version

These instructions are based on firmware version "VD22Adv_fw_1.0" or higher.

Intended use

This electronic reading device is exclusively intended for magnification and reading or viewing of texts and images. To guarantee correct functioning, the screen reader must be set up as described in the Setup guide.

The product is operated in a domestic environment (also includes nursing homes, workplaces, schools and other educational facilities).

Before first use

Please read these operating instructions thoroughly before using the device for the first time. This is the only way to get to know all the features available for the device.

Package contents

- ▶ vario DIGITAL 22 FHD Advanced
 - ▶ Remote control with 2 × AAA batteries
 - ▶ Power adapter
 - ▶ Power plug adapter for EU, GB, US, AU
 - ▶ HDMI cable
 - ▶ USB cable (Type C)
 - ▶ Microfiber cloth
 - ▶ Operating instructions
- ▶ Depending on the version:
 - incl. battery
 - incl. X-Y table

Safety information

EN

- ▶ **Risk of damage to the eye and risk of injury: Never look into the light sources on the underside of the device.**
- ▶ **Never allow children to handle electric devices without supervision.**
- ▶ **Risk of suffocation and risk of accidental swallowing: Keep packaging, films and packaging parts away from children.**
- ▶ **Risk of pinching: ensure that your fingers do not get pinched during unfolding and folding of the device, and also during unfolding and folding of the camera on it.**
- ▶ **The entire device must stand on a stable surface.**
- ▶ **Take care not to leave any objects on the base when you want to fold the device together. This can result in damage to the display.**
- ▶ **Protect the device against impact or shock. Handle the device with care.**
- ▶ **Never subject the device to humidity or moisture.**
- ▶ **Protect the device from excessive heat and open flames. Never place the device on radiators or in direct sunlight.**
- ▶ **Do not store the device in locations with excessive heat, for example in a parked car.**
- ▶ **Use only the supplied power adapter and country-specific power plug adapter.**
- ▶ **Check the device, the power adapter and its cable for visible damage. If there is any damage, disconnect the power adapter from the power supply immediately. Have the product checked by the manufacturer or his customer service before you use it again.**
- ▶ **Class 1 laser in acc. w. EN 60825-1:2014.**
- ▶ **Route the power cord so that it does not present a tripping hazard.**
- ▶ **Never open the device housing. Opening the housing voids the warranty.**
- ▶ **The device must be turned off and the power plug must be removed before it is folded together.**

- ▶ **Carry the device only when it is folded together and by using the carrying handle.**
- ▶ **Only for domestic use.**
- ▶ **In a medical environment this device may not be operated together with other devices that are used for life support.**
- ▶ **Ensure that other people, especially children and persons who cannot assess this hazard, are aware of this hazard.**
- ▶ **The product is a Class I medical device (non-sterile, without measuring function) and complies with Regulation (EU) 2017/745. For use of the product outside the EU member states, the respective country-specific directives must be observed.**
- ▶ **If you purchased the device including batteries, they may only be removed or exchanged by the manufacturer or its customer service department.**
- ▶ **CAUTION risk of explosion! Never replace the battery with the wrong type. Never dispose of a battery in a fire. Never crush or cut a battery. Never leave a battery laying around in an environment with extremely high temperatures, as this can result in an explosion or cause flammable liquids or gases to escape. Never subject a battery to extremely low air pressures, as this can result in an explosion or cause flammable liquids or gases to escape.**
- ▶ **When using the variant with X-Y table:**
 - **Make sure not to get your fingers caught when moving the table.**
 - **Only ever carry the device on the intended carrying handle when it is folded together. Fasten both brakes and press the button to lock the X-Y table (see page 25).**

Functional elements

(For illustrations see pages A1 and A2)

EN

1. Display
2. Rotary knob to open contrast functions
3. Photo button for saving images and confirming menu options
4. Button for turning the reading line on and off
5. On/Off button
6. Button for opening menu operation
7. Button for overview display
8. Rotary knob to increase or decrease zoom factor
9. Release button on the base arm
10. SD card (insert card as shown)
11. SD card slot
12. USB port
13. RESET button for resetting the device
14. Monitor arm
15. Button to move right in the menu
16. Button to move left in the menu
17. Button to move up in the menu
18. Button to move down in the menu
19. Foldable camera cover (flipped open in illustration)
20. Camera
21. Base plate
22. Release button on the monitor arm
23. Base arm for unfolding the device
24. Charging control LED
25. Connection for power adapter
26. Carrying handle
27. HDMI input

Basic operation

Structure of the device and power supply

To avoid damage to the product, follow the assembly instructions, which can be found in the product packaging. Always set up the device on a suitable surface only, e.g. a table.

The camera ⑳ is equipped with a cover ⑲. Open the camera cover and move it to the intended position (see illustration on fold-out pages A1 and A2). If the  icon is shown, the camera cover has not been opened.

Connect the power adapter to a suitable device plug. Connect its cable to the connection for the power adapter ㉕. Then plug the power adapter into the power socket. The charging control LED ㉔ lights up while the device is being charged.

To fold the device, follow the assembly instructions, which can be found in the product packaging.

You can adjust the height and angle directly on the monitor, which is suspended from a rail. Due to the locking mechanism, the angle of the arm is always the same.

Turning on/off

Press the On/Off button ⑤. The camera image will appear on the screen ① after a few seconds. Symbols briefly show the current status, for example magnification factor and brightness setting.

To turn the device off, press and hold the On/Off button ⑤ for one second. Current settings are saved.

Reading with the device

Turn the device on and place the reading material on the base plate ㉑ under the camera ㉒.

In exceptional cases it can make sense to relocate the field of vision to the upper area of the base plate. To do this, swivel the camera out past the locking position. It is also possible to move the camera forwards, for example for carrying out handicrafts.

Adapting the magnification

EN

To increase the magnification turn the rotary knob ⑧ in clockwise direction when the live image is displayed. To decrease the magnification, turn the rotary knob ⑧ counterclockwise. The magnification factor you have set will be displayed briefly.

Overview function

If you would like to see an overview of the reading material, then press the  button ⑦. The magnification will change to 1.6x. If you press the  button again, the previously set magnification will be selected again.

Dynamic Line Scrolling (DLS)

With the DLS function, you can move the image selection horizontally or vertically within the visible area without having to move the reading material.

To activate the DLS function, press and hold the  button ③ for approx. 3 seconds. The device then zooms to the preset width. That is the field of vision available for DLS. Now you can adapt the zoom factor by using the ⑧ knob.

With buttons ⑮, ⑯, ⑰, and ⑱ you can move the displayed section horizontally or vertically without having to move the reading material itself. The image selection can be moved upward with arrow key ⑫, downward with arrow key ⑬, to the right with arrow key ⑯, and to the left with the arrow key ⑰. If you press one of the arrow keys twice in rapid succession, the screen section is moved to the respective end. For example, you can quickly find the beginning of the next line by pressing the ⑰ arrow key twice.

To stop the DLS function, press and hold the  button ③. The  icon appears. Thereafter, the entire magnification range is available again.

There are three reading modes: Letter, Book und Column. You should select the mode which best matches the width of your reading material. The reading modes can be configured in the menu. The following reading modes are available, offering different magnification presets:

- ▶  Reading Mode 1: Letter (2.2 times – maximum 12 times magnification)
Use this reading mode for very wide objects (up to 215 mm/8.5")

- ▶  Reading Mode 2: Book (3.3 times – maximum 28 times magnification)
Select this mode to read a book (width based on reading mode) using DLS.
- ▶  Reading Mode 3: Column (6 times – maximum 48 times magnification)
To read (newspaper/magazine) columns using DLS, select this mode.

DLS is always deactivated when the device is turned on.

Alternatively, the DLS can also be operated using the remote control (see “Remote control” chapter on page 22).

Turning the reading line/row on and off

Press the **4** button to turn the reading line on. Press it again to hide the line.

Special button functions

- ▶ Press and hold the  button **3** to activate/deactivate DLS (see “Dynamic Line Scrolling (DLS)” chapter on page 20).
- ▶ Press and hold the  button **7** to turn the autofocus function off (see also Chapter “Deactivating/activating autofocus” on page 22).
- ▶ Press and hold the reading line button **4** to change the reading material brightness (see “Adjusting the reading material brightness”).

False color display

To display the camera image in false colors for increased contrast, turn rotary knob **2** until the required false color rendering is displayed. By default, the false colors are preset as black/white and white/black. To adapt the false color display, open the “Adjusting false colors” menu.

Saving the live image as a file

Press the  button **3** briefly to save the current camera image as a file on the inserted SD card and to show it as a still image on the screen. You can also change the magnification factor for this image as already described. You can use the arrow keys **17** + **18** to move the magnified image section vertically, and arrow keys **15** + **16** to move it horizontally. Press the  button again to return to the live image display.

Deactivating/activating autofocus

Press and hold the  button **7** to deactivate the autofocus function. This can be helpful, for example while writing under the device, so that the focus remains on the reading material. Press the  button again to reactivate autofocus. Autofocus is automatically reactivated when the device is turned off.

Remote control

The remote control can be used while you are holding it in your hand and also when it is lying on the table. The anti-slip strips ensure that the remote control grips the table well. The remote control only works if there is a visual connection between yourself and the IR sensor below the display.

Setting-up for use

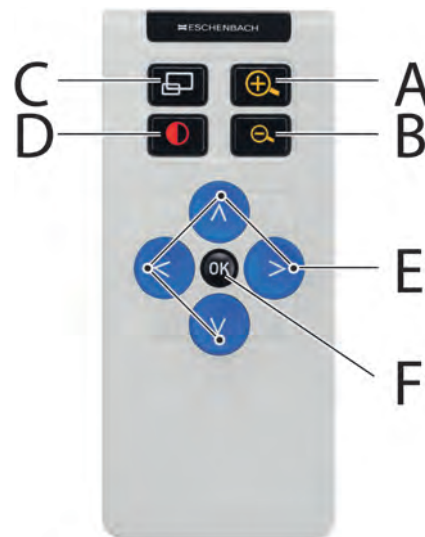
One battery terminal is insulated against the contact area with a plastic strip to avoid self-discharge. To remove it, open the battery compartment cover on the underside of the remote control and pull the strip out.

Adapting the zoom

To zoom in, press **A**. To zoom out, press **B**. The currently selected zoom factor is shown briefly in the display.

Dynamic Line Scrolling

To use Dynamic Line Scrolling, press and hold the OK button **F** briefly. Once the desired zoom factor has been set, you can use the arrows to move the displayed section horizontally or vertically without having to move the reading material itself. If you press the arrow buttons at the end/beginning of the image selection for 2 seconds, the image selection jumps to the other end/beginning. This makes it much easier to read text.

Overview function

If you would like to get an overview of the reading material, press  button **C**. The magnification changes to 1.6 x. By pressing the  button **C** again, the previously set magnification is selected again.

False color display

To change false colors, press the **D** button until the desired false color is displayed.

Channels

A switch for switching over the channels is located in the battery housing. However, this is only necessary if two devices are being used next to each other. You can set the relevant channel in the menu under "Remote control".

The channel selected on the remote control and in the menu must be identical. Only then can the remote control be used. Channel 1 is used by default.

The remaining description of the menu can be found under Quick Guide, the first menu item on the device. Press the MENU button ⑥ to open the menu.

Connections

USB port

You can use the supplied USB cable to connect the device to the USB port on a computer using the device USB port ⑫. The SD card inserted in the device will then be displayed on the computer as the "ESCHENBACH" drive for file operations.

HDMI

You can use the enclosed HDMI cable to connect the device via its HDMI input ⑳ to the HDMI output of other devices. If the device detects the HDMI signal at the input, it automatically switches to the display of this signal. To switch back to the camera display, press and hold the MENU button ⑥ for 3 seconds. To switch back to the HDMI signal, press and hold the MENU button ⑥ for 3 seconds again. If the connection to the HDMI device is interrupted, the device automatically switches to the camera display.

This function depends on the output device. The camcorder or the camera must be fitted with an HDMI output and HDMI Live.

EN

If you do not need sound, you can set this in the menu. Open the menu and set the volume () to muted ()

Reset

If the device no longer functions as described in these instructions, it may be due to a malfunction, which you can eliminate yourself by resetting the device. Then press the RESET button .

Additional equipment variants

Battery

If you own the vario DIGITAL 22 FHD Advanced with a battery, please read this paragraph carefully:

- ▶ The battery may only be removed or exchanged by the manufacturer or his customer service.
- ▶ It takes 4 hours to completely charge the battery, and this allows an operational duration of 2.5 hours.

X-Y table

If you own the vario DIGITAL 22 FHD Advanced with X-Y table, please read this paragraph carefully:

28. Horizontal brake (unbraked)

29. Vertical brake (unbraked)

30. Button for locking the X-Y table

31. Button for releasing the lock

Before you can use the X-Y function, press the button to release the lock 31. Then, move the brakes 28 and 29 to the shown position.

You can now move the X-Y table horizontally and vertically.

If you only move brake 28 right, the X-Y table can only be moved vertically. If you only move brake 29 up, the X-Y table can only be moved horizontally. For transport, only ever carry the device on the intended carrying handle 26 when it is folded together. Fasten both brakes 28 and 29 and press the button to lock the X-Y table 31.

Supplier's Declaration of Conformity

47 CFR § 2.1077 Compliance Information

EN

Unique Identifier

Trade Name: vario DIGITAL FHD

Model No.: 16592, 165922, 1659221, 1659222, 1659223

Responsible Party – U.S. Contact Information

Eschenbach Optik of America, Inc.

22 Shelter Rock Lane

Danbury, CT 06810

United States

Tel.: (800) 487-5389

Fax: (888) 799-7200

FCC Compliance Statement

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- ▶ Reorient or relocate the receiving antenna.
- ▶ Increase the separation between the equipment and receiver.
- ▶ Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- ▶ Consult the dealer or an experienced radio/TV technician for help.

MODIFICATION: Any changes or modifications not expressly approved by the grantee of this device could void the user's authority to operate the device.

Technical specifications

Display size:	54,6 cm (21,5") 16:9 Full HD
Camera:	optical/digital zoom camera with AF
Magnification range:	1.6 × - 100 ×
Display modes:	real color, contrasting up to 14 false color combinations possible
Card formats supported:	SD, SDHC (max. 32 GB)
Charger power supply:	Primary: 100 V – 240 V ~ 50 – 60 Hz Secondary: 19 V $\equiv$ 2000 mA
USB port:	Type C (only for data transfer)
HDMI input:	HDMI 1.4, depending on output device
Operating temperature range:	+10 °C – +40 °C (50 °F – 104 °F), humidity: < 80 %
Storage temperature range:	0 °C – +60 °C (32 °F – 140 °F), humidity: < 80 %
Dimensions:	520 × 150 × 500 mm (20.5" × 5.9" × 19.7") (W × H × D)
Dimensions with X-Y table:	520 × 180 × 530 mm (20.5" × 7.1" × 20.9") (W × H × D)
Weight:	w/o X-Y table: 8,9 kg incl. X-Y table: 11,6 kg
Lighting:	LED
Power consumption:	~ 30 W
Images per second.:	60 fps
Contrast ratio:	1000 : 1

These instructions are also available on the Internet at www.eschenbach.com

USB® is a registered trademark of USB Implementers Forum, Inc.

Care instructions

EN

- ▶ Turn off the device and disconnect it from the power source before you clean it.
- ▶ Clean the device using a soft, dry cloth only.
- ▶ Never use soap solutions containing plasticizers or alcoholic or organic solvents to clean your device, and never use abrasive detergents.
- ▶ Never clean the device in an ultrasonic bath or under running water.
- ▶ Protect your device against heavy contamination.
- ▶ Condensate may form and cloud the lens when you move from cold rooms to warm rooms. If this happens, wait 40 minutes before starting the device to avoid condensate formation.

Disposal

- ▶ This product is subject to the provisions of European WEEE Directive covering old electrical and electronic devices in its currently applicable version. Never dispose of your old device in normal domestic waste. Instead dispose of it through an approved disposal facility or your municipal waste facility. Observe the regulations applicable in your country. In case of doubt, contact your waste disposal facility. Dispose of all packaging materials in an environmentally friendly manner.

Warranty

Within the parameters of the applicable statutory provisions, we provide a warranty for the functioning of the product described in this manual with regard to faults arising in connection with manufacturing defects or material defects. No warranty claims can be accepted for damage that arises due to improper handling, including damage from falls or impacts. Warranty claims can only be accepted on presentation of the sale receipt.

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USB® es una marca registrada de USB Implementers Forum, Inc.